

E-XCLUSIF MEDICAL SUPPLY PTE LTD

TEL: 6738 3525

IMPORTANT NOTE FOR CARE OF NEBULIZERS

Please DETACH the plastic tubing from the nebulizer and the medicine container after use. Coil the plastic tubing neatly on top of the nebulizer when keeping it in the nebulizer box/bag or hang the tube loosely in a clean and airy area when the nebulizer is not in use. This is to prevent the tubing from being kinked.

Kinked tubing will cause the backflow of air into the nebulizer compressor and this will slow down the performance of the nebulizer drastically and will eventually cause the nebulizer to breakdown.

The nebulizer medicine container and the mask are to be washed after use and the medicine container, tubing and mask (nebulizer kit set) has to be changed once every 1-2 months even when not in use. Please contact us if you require clarification or if you need to purchase the nebulizing kit set.